

MEMO

DATE: October 4, 2021

TO: All SHA Managers

FROM: SMART Pump Program

RE: LifeCare PCA 7 Infusion Pump – Change to Cleaning and Disinfecting Requirements

Beginning the week of October 18 2021, all LifeCare PCA 5 pumps will be switched out for LifeCare PCA 7 infusion pumps (LPCA 7 pumps).

ICU Medical requires **specific cleaning products** to be used on these new pumps.

Approved cleaning and disinfecting agents include:

- Clorox™ Germicidal Bleach (8.25% concentration) – Do not exceed 1 part bleach in 10 parts water
- ASP Enzo™
- ECOLAB OxyCide™
- Johnson & Johnson/ASP® CIDEX® OPA

ICU Medical will not support the use of any other agents for cleaning and disinfecting the new LPCA 7 Infusion Pumps. This means the warranty will be void if these pumps are cleaned or disinfected with any other cleaning or disinfecting agent.

Please ensure your department is using only the listed cleaning and disinfecting solutions to clean LPCA 7 Pumps.

Thank you for your cooperation with this updated practice.

Please be in touch with the Smartpump Program if you have any questions or concerns about this change.

Tara Griffiths, RN
Manager, Provincial Clinical Programs
Clinical Standards & Professional Practice

Safety | Accountability | Respect | Collaboration | Compassion

With a commitment to a philosophy of Patient and Family Centred Care

NOTICE OF CONFIDENTIALITY: This information is for the recipient(s) listed and is considered confidential by law.
If you are not the intended recipient, any use, disclosure, copying or communication of the contents is strictly prohibited.